



## WHO WE ARE

RD Keegan, DVM, Diplomate, ACVAA  
JA Noyes, DVM, MA (Ed Psych)  
Clinician educators dedicated to improving  
student learning and retention



# VETERINARY SIMULATION ENVIRONMENT

[www.wholelogic.com](http://www.wholelogic.com)

**“The technology is  
groundbreaking and so vital”**

Stage III places learners in a low risk, realistic  
environment providing dynamic learning  
through adaptive and interactive technology




**STAGE III**  
by WholeLogic


**Designed with clinical experience**  
**Refined with educational theory**  
**Validated with educational research**

- Facilitates Deliberate practice and inquiry-based learning
- Promotes decision making and problem solving
- Improves clinical skills, knowledge and self efficacy
- Provides explicit explanatory feedback.

## GET IN TOUCH

 [admin@wholelogic.com](mailto:admin@wholelogic.com)  
[www.wholelogic.com](http://www.wholelogic.com)


 WholeLogic Inc.  
Viola, ID, USA

 (208)-820-4231




## HOW DOES IT WORK?

- The Stage III simulation software runs on two display screens, one controlled by an instructor and a second visualized by simulation students as a vital signs monitor.
- Use your PC's extended display setting and drag the student display onto the external monitor.
- Changes made on the instructor screen or by the scenario programming are displayed on the students vital signs monitor.

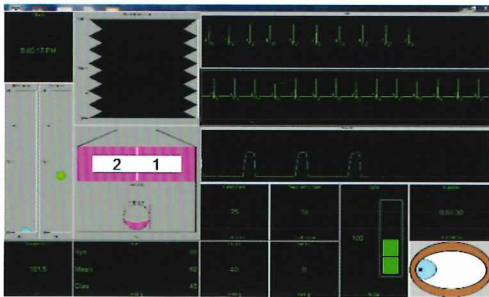

Stage III Simulation Display

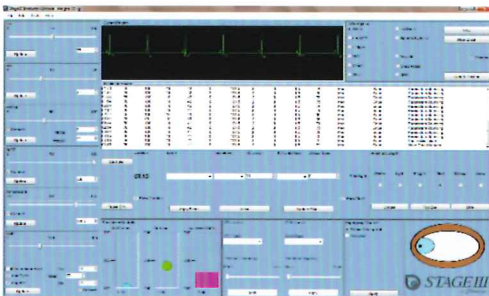

Stage III Instructor Display

## STAGE III BRINGS IMMERSIVE SIMULATION TO VETERINARY MEDICINE.

Developed by veterinary specialist/educators at Washington State University and extensively tested, Stage III is an immersive simulation environment aligning with the educational needs and fiscal realities of veterinary medical training

“

*The simulation allowed me to practice anesthetic monitoring in risk free but very life like environment.*

”

## WHAT'S REQUIRED?

- Requires a desktop/laptop PC and an external monitor, that's it!
- BYOM - Any manikin will work with Stage III including a manikin you create so feel free to Bring Your Own Manikin
- Add equipment that you probably already have (anesthesia machine, monitor, surgery table) to enhance realism.

## GETTING STARTED IS EASY... JUST LAUNCH, DRAG AND CLICK

Launch the simulation, drag to the student monitor and click to start

Run a programmed scenario or adjust parameters on the fly

Save the simulation history for debriefing

Stage III at IVECCS 2018

